

BreathTracker Maintenance Matrix

THE BREATHTRACKER MAINTENANCE MATRIX CAN BE USED TO HELP END-USERS DETERMINE THE MAINTENANCE PRODUCTS UTILIZED IN BREATHTRACKER SYSTEMS.

BreathTracker MODELS

CATALOG # & DESCRIPTION	SC	DP	H2+	H2	Additional Info.
QT01154-C - SivRite-4 Desiccant	X	X	X	X	Attaches to Air-Intake port on the BreathTracker, replace when beads change color indicated on label.
QT01156-C - 10/20 Mesh Indicating Drierite®	X	X	X	X	Used to remove water vapor in patient samples via Patient Sample Drying Tube. Replace contents when 3/4 pink.
QT07210-G - QuinGas-1 (~150ppm H ₂)				X	Reorder when psi reaches below 50psi.*
QT07220-G - QuinGas-2 (~150ppm H ₂ , 75ppm CH ₄)		X			Reorder when psi reaches below 50psi.*
QT07230-G - QuinGas-3 (~150ppm H ₂ , 75ppm CH ₄ , 6% CO ₂)	X				Reorder when psi reaches below 50psi.*
QT07008-G - QuinGauge, Pressure Gauge	X	X	X	X	Used to check psi levels in QuinGas cylinders, replace if lost or broken.
QT02592 - QuinGas Valve Removal Tool	X	X	X	X	Used to disengage valve on QuinGas cylinders for disposal.
QT01140-K - Water Barrier/Dust Trap (Package of 5)	X	X	X	X	Should be replaced at least once every 6 months or if dirty/discolored.
QT01135-K - Patient Sample Drying Tube	X	X	X	X	Filled with Drierite, and 2-Foam Filter Plugs. Used in-line with instrument and patient sample.
QT00527-T - Foam Filter Plugs	X	X	X	X	Used with Patient Sample Drying Tube and should be replaced periodically.
QT01741 - 30mL Bulk Non-Sterile Syringe	X	X	X	X	Used for injection of QuinGas into instrument or transferring of samples.
QT01735-V - 35mL Monoject Plastic Syringe	X	X	X	X	Used for injection of QuinGas into instrument or transferring of samples.
QT01760-V - 60mL Monoject Plastic Syringe	X	X	X	X	Used for injection of QuinGas into instrument or transferring of samples.

All products listed above do not need to be replaced when analyzing multiple patient samples.

Maintenance materials are used to ensure longevity of the BreathTracker system.

Failure to replace components when needed can result in:

Damage to the product(s), adversely affecting patient samples, and voiding of your instrumentation's warranty!

If there are any questions regarding these products or their use please consult with QuinTron's Customer Service Department.

*psi = pound per square inch

*ppm = parts per million

Drierite® is a registered trademark of W.A. Hammond Drierite Company, Ltd.



3712 West Pierce Street, Milwaukee, WI, USA
www.QuinTron-USA.com
Phone: (800)542-4448 / (414)645-4222